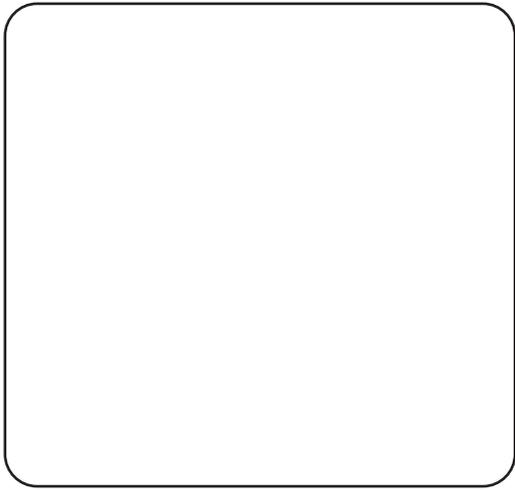
INSTRUCTIONS OF USE

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www.oxyderme.com

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Contents

1. General	3
1.1 Information in these instructions for use	3
1.2 Type plate	3
1.3 Liability and warranty	3
1.4 Warranty	4
1.5 Explanation of symbols	4
1.6 Customer service	5
2. Safety	6
2.1 General	6
2.2 Customer's responsibility	6
2.3 Intended use	7
2.4 Dangers which may arise from the unit	7
2.5 What to do if a hose caught fire	9
3. Design and function	10
3.1 General description	10
3.2 Design	10
4. Technical data	11
5. Transport, packaging and storage	13
5.1 Transport inspection	13
5.2 Storage	13
6. Taking into service	14
6.1 Before assembling	14
6.2 Choosing the location	14
6.3 Assembly	15
7. Operation	16
7.1 Taking into service	16
7.2 Adjusting the oxygen volume flow	17
8. Maintenance	18
8.1 Safety	18
8.2 General notes	18
8.2.1 Cleaning	18
8.2.2 Disinfecting	18
9. Appendix	19
10. Index	20

1. General

1.1 Information in these instructions for use

These instructions for use describe the installation, operation and maintenance of the unit. Strict compliance with the stated notes on safety and instructions for use is a prerequisite for safe and proper work with the unit.

This instruction manual is part of the product and should be kept near the unit so that it is at any time available for personnel involved in installation, operation, maintenance and cleaning.

The graphic illustrations in this manual may perhaps differ slightly from the actual design of the unit.

1.2 Type plate

The type plate of the Oxyderme THBO concentrator is attached to the back of the unit above the coarse dust filter.

1.3 Liability and warranty

All details and notes for the operation, maintenance and cleaning of the unit are made to the best of our knowledge taking into consideration our experience and knowledge gained up to now.

We reserve the right to make technical changes to the machine dealt with in these instructions for use within the context of ongoing development.

Translations are also carried out to the best of knowledge. We do not accept any liability for errors in translation.

Texts and illustrations do not necessarily correspond to the scope of delivery. The drawings and graphics are not to scale 1:1.

Read instructions for use carefully before starting operation of the unit!

The manufacturer will not assume liability for damage or disruptions that occur as a result of non-compliance with the instructions for use.

Handing over these instructions for use to third parties is not permitted and results in the obligation for compensation.

1.4 Warranty

We grant, in excess of the implied warranty the following extended warranty:

Warranty terms: maximum 30,000 operating hours or 5 years from the date of purchase on all functional parts (e.g. compressor, electronic circuit board, solenoid technology, etc.). The first reached condition (30.000 hours or 5 years) applies.

Condition precedent to the extended warranty is the periodic maintenance according to the Technical Service Manual.

Extent of warranty: the free-of-charge replacement of defective parts. The defective parts must be submitted to us for inspection of warranty claims. For processing the warranty claim, the serial number as well as date of purchase and the operating hours of the affected device must be stated. If devices are sent to us for warranty repair free of charge, we will also cover the labor costs for any warranty repairs. Locally resulting travel and service expenses are not covered by us.

Excluded from this warranty are: filters and zeolites, qOartz, damage from improper handling and mechanical damage of parts (e.g. transport damage).

1.5 Explanation of symbols

Important safety and equipment related notes in these instructions for use are highlighted by symbols. These notes must be strictly adhered to in order to avoid accidents, personal injuries and damage to property.

WARNING!

This symbol warns of dangers that can lead to adverse effects on health, injuries, permanent physical damage or to death.

Strictly comply with all notes regarding work safety, and be particularly careful in these situations.

WARNING! Danger of electric current!

This symbol draws attention to dangerous situations involving electrical currents.

There is a danger of serious injury or death if the safety notes are not complied with. The related work may only be carried out by qualified electricians.

ATTENTION!

Information highlighted with this symbol must be strictly complied with in order to avoid damage to the machine, malfunction and/or breakdown.

NOTE!

This symbol highlights hints and information to be observed for efficient and non-disrupted operation of the unit.

1.6 Customer service

Service work should normally be carried out by your local authorized dealer. You can contact Oxyderme THBO as follows:

Office hours:	Mo-Fri 8.30 a.m. – 6.0 p.m.
Address:	Oxyderme THBO Dumontshaff L-3841 Schiffflange Luxembourg
Phone:	+352 26 55 34 22
Internet:	http://www.oxyderme.com
Email:	sav@oxyderme.com

2. Safety

This section provides an overview over all important safety aspects for safe and trouble-free operation of the unit.

The individual chapter additionally contain actual safety notes highlighted with symbols, which will help to avoid immediate dangers.

2.1 General

The machine is built according to the currently applicable rules of technology and is safe to operate.

However, dangers may still arise from the unit if it is incorrectly operated or used for purposes it is not intended for.

Any persons using this unit must have read and understood these instructions before starting operation. This also applies if the person in question has already worked with just such a unit or similar equipment or was trained by the manufacturer.

Knowing the content of these instructions for use is a prerequisite for the avoidance of mistakes and for safe and trouble-free operation of the unit.

Neither changes nor conversions may be carried out on the equipment, which have not been explicitly authorized by the manufacturer, to avoid dangers and to ensure optimal performance.

All safety decals and operating signs on the unit must be kept well legible at all times. Damaged or illegible decals must be replaced immediately.

2.2 Customer's responsibility

These instructions for use must be kept near the unit, so that it is available for the user at any time.

Apart from the notes on safety mentioned in this manual, all generally valid safety and accident prevention instructions must also be observed and adhered to.

The machine may only be operated in a technically perfect condition and if operationally safe.

The information contained in the instruction manual is complete and must be adhered to without limitation.

2.3 Intended use

The operational safety of the unit is only assured when used for the purpose it is intended for, as specified in the instructions for use.

The Oxyderme THBO concentrator solely intended for use within the scope of a non-life sustaining anti-aging therapy. The unit may thereby be used in beauty shops, spas, hotels or doctor's offices.

The unit must only be used according to the Oxyderme protocol and only by following these instructions for use.

The intended use also includes the compliance with the assembly instructions, as well as the notes on cleaning and maintenance of the unit.

Any other use of the unit beyond these limits is prohibited and is not considered as unintended use! Claims of any kind against the manufacturer and/or his authorized representatives resulting from damage caused by unintended use of the unit are excluded. The customer is solely liable for any damage resulting from unintended use.

2.4 Dangers which may arise from the unit

The unit was subjected to a risk analysis. The resultant construction and design of the unit corresponds to the current status of technology.

However, there is still a remaining risk!

The unit requires responsibly minded and cautious operation. Improper operation or operation by unauthorized persons can endanger persons.

WARNING! Risk of health damage!

If the customer or the operator notices at any time that the available amount of oxygen is not sufficient, you should immediately contact your dealer.

WARNING! Risk of health damage!

Particular supervision is required if the unit is to be used in the vicinity of children or bedfast persons. The unit must under no circumstances be used with children without additional supervision!

WARNING! Danger of side effects!

Do not modify your device. You can endanger your health. If side effects or extreme health restrictions occur during the therapy, you should immediately consult your doctor.

WARNING! Fire hazard caused by oxygen!

Oxygen is vital, but in concentrators with only a few percentage points above the normal oxygen content in the air it is a highly dangerous fire accelerant. There are only a few materials which will not burn off like an explosion under a raised concentration of oxygen.

Therefore:

- Oxygen may only be handled by trained or specially instructed persons!
- Follow all mandatory instructions for installing and use of fire-reducing equipment such as the angular connector and Firesafe™ check valve.
- The misuse of oxygen, e.g. to cool down or improve the ambient air, to cool down and dust or blow off of persons, clothes, furniture etc. is dangerous and therefore prohibited!
- Smoking and handling ignition sources and open flames is prohibited when working with oxygen!
- Keep a minimum distance of 2 metres from all spark generating equipment and open fire!
- After having stayed in a possible oxygen saturated atmosphere you should thoroughly aerate your cloths, because oxygen adheres to the clothes very well! An ignition source, e.g. a burning cigarette, could easily cause burning of your clothes.
- Materials that do not burn in air may burn very vigorously and even spontaneously in oxygen or oxygen enriched air. This already applies for an enrichment of only a few percent!
- Oil and grease (even creams and gels) can react in an explosion like manner when coming in contact with oxygen. It is therefore mandatory to keep the unit free of oil and grease!
- Oxygen considerably increases the temperature of a flame and the speed of combustion!

**WARNING! Danger of electric current!**

Electric energies can cause severe injury. Damaged insulation or components cause a danger to life.

Therefore:

- Work on the equipment must only be performed by trained professionals.
- Pull the main plug out before starting work on the unit!
- Check main leads for damage before every use.

**ATTENTION! Observe high frequency safety!**

Such equipment can be influenced by (mobile) HF communication equipment (e.g. mobile phones).

Do not use mobile radio equipment in the vicinity of the Oxyderme THBO concentrator as close as 3.5 m.

**ATTENTION! Observe electromagnetic compatibility!**

Electrical equipment is subject to stringent protective measures concerning electromagnetic compatibility (EMC) and must be installed and operated in accordance with the EMC information contained in the accompanying documents. The following should be noted in particular:

- Floors should be made of wood or concrete or should be covered with ceramic tiles. If the floor is covered with a synthetic material, the relative humidity must be at least 30%.
- During operation the unit must therefore not be exposed to extremely strong magnetic fields.
- Magnetic fields at main frequency must comply with the typical values found in business or hospital environments.

**ATTENTION! Keep the minimum distance!**

The air intake of the Oxyderme THBO concentrator is located on the back side of the unit, the following is therefore of importance:

- There should be a minimum distance of 30cm from walls, curtains and other large objects (e.g. cupboards), so that an unrestricted entry of air is assured at the back of the unit.
- The Oxyderme THBO oxygen concentrator must not be used directly beside or even stacked with other equipment.

**ATTENTION! Avoid overheating!**

The unit is air cooled, in order to avoid overheating it must not be placed near heaters, etc.

2.5 What to do if a hose caught fire

Should a hose catch fire despite all protective measures, it is not enough to just switch the unit off, because oxygen will continue to flow for a while after the unit has been switched off.

Take the following steps:

- Pull the oxygen hose off the unit to interrupt the oxygen supply.
- Choke the flames (e.g. with a blanket).
- Ventilate well after extinguishing the fire, because a burning PVC-hose will emit toxic gases.

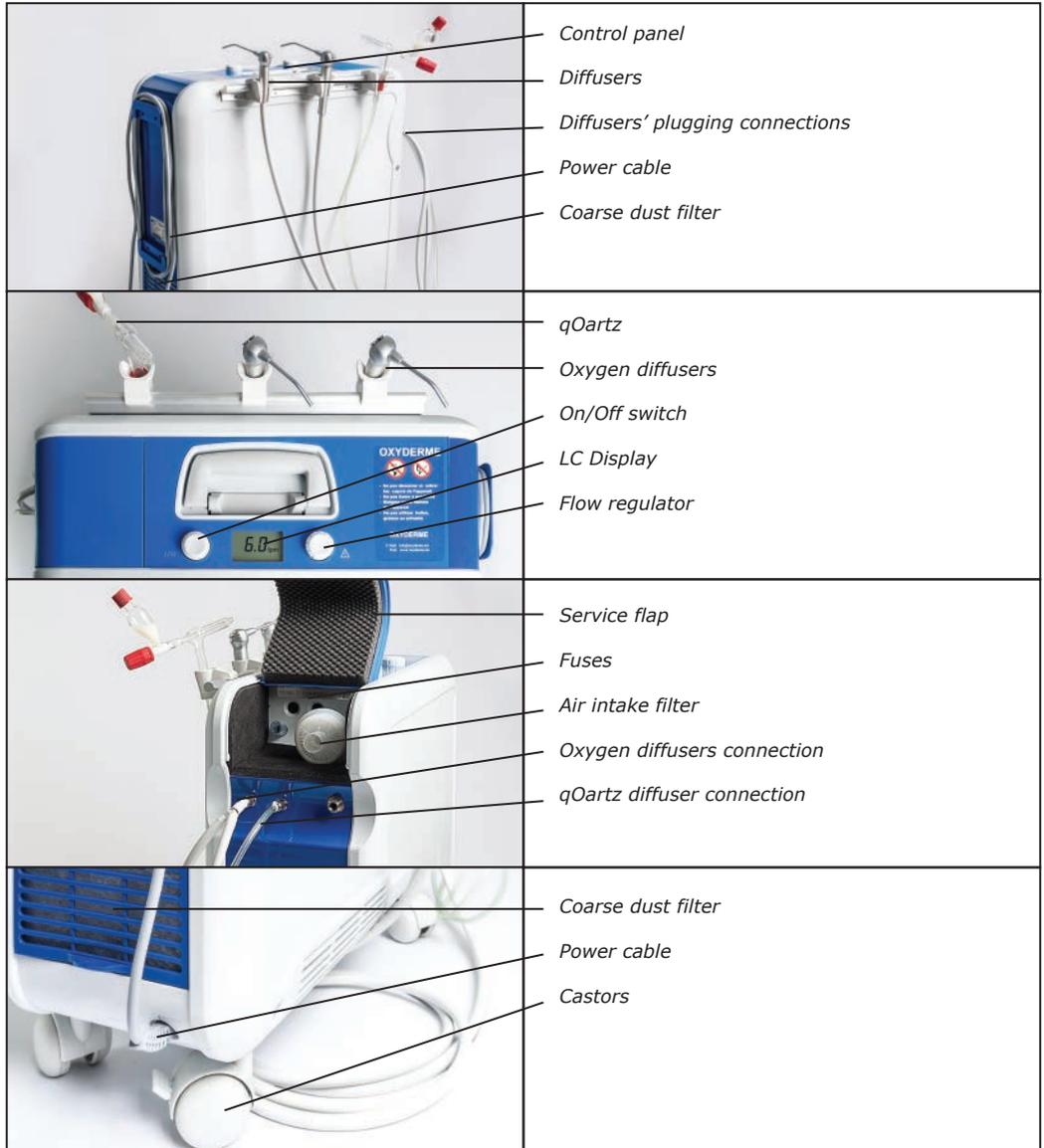
The metal connection on the oxygen outlet acts as a fire inhibitor, preventing the flames from spreading into the unit.

3. Design and function

3.1 General description

The Oxyderme THBO oxygen concentrator has been optimized for the oxygen supply at home. The electronically controlled oxygen concentrator separates the oxygen from the ambient air and supplies the user through the diffusers with highly concentrated oxygen.

3.2 Design

4. Technical data


Model	Oxyderme THBO
Classification acc. to MPG	IIa
Operating voltage	230 V, 50 Hz
Ambient operating conditions	Operation temperature: +5°C to +40°C Storage temperature: -25°C to +70°C Humidity: 15% - 93% r.H., non-condensing Atmospheric pressure: 700 to 1,060 mbar
Sound pressure level	35 dB(A)
Power input	360 W
Coarse dust filter	in back of housing
Bacteria filter	behind service flap
Fuses	main side: 2 x TT2.5 AH 250 V internal: 1 x T1.0 AL 250 V
Interface	USB  <i>WARNING! Unsafe connections! Connect only manufacturer-authorized devices to this USB port.</i>
Weight	19.8 kg
Dimensions (H x W x L)	53.5 x 20.3 x 52 cm, without castors
Warranty	30,000 hours, maximum 5 years (see chapter 1.4)
O ₂ -concentration	1 to 4 l/min. 95% - 3% 4 to 5 l/min. 85% +/- 3% 5 to 6 l/min. 75% +/- 3%
Status display O ₂ (at operating temp.)	82% status display 60% deficiency display
Max. recommended vol flow	6 l/min.
Volume flow (at 0 or 7kPa)	1 - 6 l/min. acc. to pre-settings
Max. output pressure	70 kPa

Overpressure relief

200 kPa (oxygen reservoir)

250 kPa (compressor)

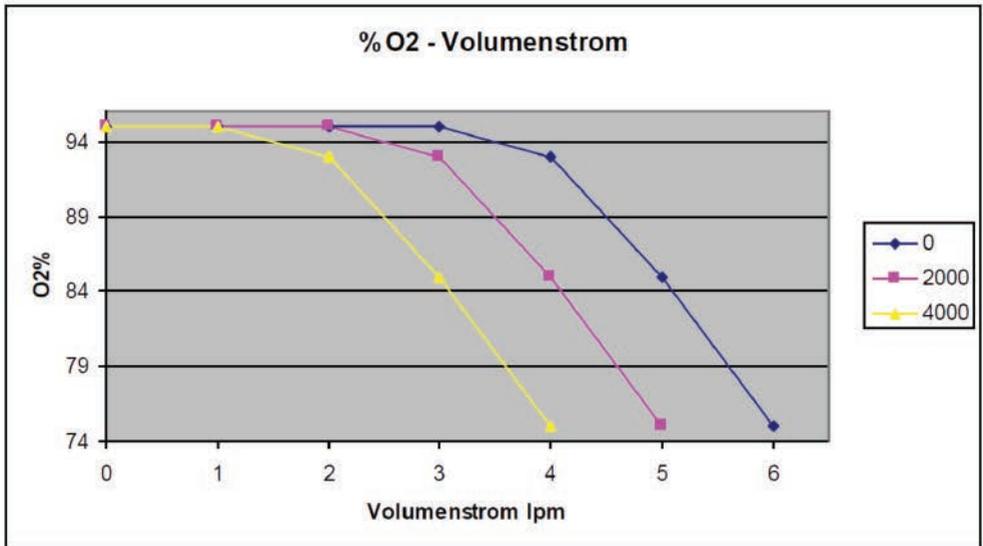O₂% characteristics, at a counter pressure of 0 kPa, O₂ concentration in %, dependent from height (m):

5. Transport, packaging and storage

The following should be noted when transporting the Oxyderme THBO oxygen concentrator:

- The unit should only be shipped and transported in its original packaging.
- For transport, e.g. by car, the unit may stand upright or lay on one of the two large flat sides.
- Open the transport box from the top. Do not stand the transport carton upside down or on one of its narrow sides.

5.1 Transport inspection

It is highly recommended to check the complete delivery for completeness and possible transport damage, immediately after receipt.

In case of externally detectable transport damage you should not accept the delivery, or only with reservation. Acknowledge the receipt only with reservation (e.g. on the freight document). Specify the expected damage and inform the manufacturer immediately.

Hidden damage should be claimed immediately after detection, because damage claims can only be lodged within the applicable claims periods.

The packaging material should be saved, it may be needed if the unit has to be returned.

5.2 Storage

If the package is to be stored before it is taken into service, please observe the following instructions:

- Store in a dry environment. Relative humidity: max. 93 % without condensation.
- It must be assured that the package is not stored outdoors. It must also be assured that the floor used for storage is dry over the entire storage period.
- Storage temperature -25 to +70°C.
- Store in a dust-free environment.
- Avoid mechanical shocks and damages.

6. Taking into service

6.1 Before assembling

Check before assembling whether all components needed for correct operation are available.

NOTE!

Contact the manufacturer or the local service provider if help is needed during operation or maintenance. The address of the manufacturer can be found on chapter 1.6. The address of the manufacturer should have been disclosed during initial training on the device.

Box content:

- Oxyderme THBO concentrator unit.
- User manual
- Coarse dust filter.
- Fire safe check valves.
- Oxygen safety hose 15m.
- Oxygen safety hose 0.33m.

6.2 Choosing the location

Please consider the following when choosing the location:

- The unit should have 30cm clearance from any walls, curtains and other large objects (e.g. cupboards), to ensure unrestricted entry of air through the back of the unit.
- The unit is air cooled. It must therefore not be placed near heaters, etc. At such a location there is a risk of overheating.
- The unit should be operated only in places where it is not impaired by air pollution or smoke.

NOTE!

The unit can be safely lifted and moved by the carrier handle on top of the unit.

ATTENTION! *Ensure a sufficient air supply!*

Ensure that the air inlet of the Oxyderme THBO oxygen concentrator is located in a well-vented spot.

The Oxyderme THBO oxygen concentrator must not be operated directly next to or even stacked with other equipment. Ensure a sufficient distance from the walls, etc.!

6.3 Assembly

Insert the power cable plug into a professionally grounded socket.

ATTENTION!

The Oxyderme THBO oxygen concentrator is only designed for operation on a 230 Volt, 50Hz electric network.

Connect the oxygen hose to the outlet connector on one side and to the diffusers on the other side.

Then connect the whole to the oxygen outlet of the concentrator.

Connect the hyaluronic acid hose to the outlet connector on one side and to the qOartz diffuser on the other side.

Then connect the whole to the oxygen outlet of the concentrator.

WARNING! Danger of tripping over!

On the back of the Oxyderme THBO oxygen concentrator you will find a winding device for the mains lead.

This should be used when the unit is out of use, to avoid any danger of tripping over.

7. Operation

WARNING! Health risk!

Inappropriate use of the Oxyderme THBO concentrator can lead to severe personal and/or material damage.

You should therefore only start up the unit in strict compliance with the instructions for use and the notes on safety.

7.1 Taking into service

Start the unit by activating the mains switch (1) on top of the unit.

- The Oxyderme THBO concentrator runs a self-test.
- The oxygen production starts after the self-test.
- The LC-display continuously shows the current oxygen volume flow and the equipment status.

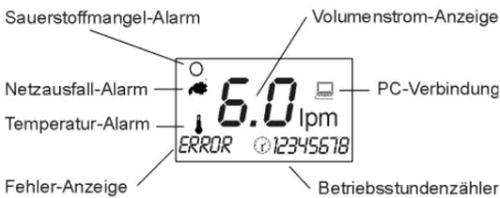
NOTE!

During the start-up phase the oxygen alarm appears for 2 minutes. This alarm should disappear after two minutes. If this does not happen, the unit may be defective.

After maximum 5 min, the unit will reach its complete performance according to the Technical Specifications.

During the start-up phase the unit also shows the operating hours. This information disappears after 30 seconds.

Information concerning the operating status of the unit can be found in the display.

WARNING! Risk of health damage!

Do not smoke while operating the Oxyderme THBO concentrator!

Oxygen must NOT be used near open fire, sparks, glowing objects, etc.!

7.2 Adjusting the oxygen volume flow

1. Press the adjustment regulator (2) for at least 2 seconds.
- When a signal sounds and the volume flow indication in the LC display flashes the adjustment mode has been activated.

2. Change the volume flow by turning the adjustment regulator. Turning clockwise increases the volume flow; turning the regulator anti-clockwise reduces the volume flow.

NOTE!

Perform the following steps to adjust:

0 - 2 l/min in steps of 0.1

2 - 4 l/min in steps of 0.2

4 - 6 l/min in steps of 0.5

3. The settings are accepted by pressing the adjustment regulator again. The input is once again acknowledged by an audible signal.

NOTE!

The supplier may have restricted the available volume flow range to customized settings, such as 1 -4 lpm.

NOTE!

It may take up to 2 minutes until the actual volume flow corresponds with the setting. If the output is closed, the output pressure can increase up to 0.7bar. The maximum gas output temperature is maximum 6 degree higher than the ambient temperature.

NOTE!

After adjusting the oxygen flow setting, the alarms and status messages for oxygen concentration and volume flow deviations are deactivated for approximately one minute until the unit has reached a steady state. During this period alarms are muted.

ATTENTION!

The unit has been designed for operation at altitudes of up to 2,000 meters above sea level. If the unit is used beyond this specification, compliance with the specified performance data can no longer be guaranteed.

8. Maintenance

8.1 Safety

WARNING! *Danger of electric current!*

Before starting cleaning the unit must be switched off and disconnected from the mains supply.

8.2 General notes

Cleanliness is a prerequisite for the success of an oxygen anti-aging therapy. The specified cleaning intervals must therefore strictly adhered to!

8.2.1 Cleaning

- The unit should be cleaned with a damp (not wet) cloth, so that not fluid can enter.
- You should only use commercial cleansing agents (e.g. washing-up liquid).
- Aggressive cleansers must not be used under any condition!

8.2.2 Disinfecting

- Any commercial disinfectant can be used for disinfecting. An up-to-date list is available from the manufacturer.
- The information for use issued by the disinfectant manufacturer must be strictly complied with.

9. Appendix

Symbol	Explanation
	Attention, observe instructions for use!
	Attention, observe instructions for use!
	Class II
I/O	On / Off
	Do not smoke!
	No open fire!
	No oil or grease!
	Do not remove any covers!
	Do not dispose of in household waste!

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